EXION™

The Ultimate Skin Solution

EXION™

# TAMAMI TEK CİHAZDA

**SONSUZ FIRSAT**

**The Ultimate Skin Solution**

**EXION™**

# 4 ÖZEL EXION APLİKATÖRÜ

**Fractional RF**

**Face**

**Body**

**EMFEMME 360°**

**The Ultimate Skin Solution**

**EXION™**

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BEFORE

AFTER EXION

1 MONTH AFTER THE LAST TREATMENT, COURTESY OF: DIANE DUNCAN M.D.

BEFORE

AFTER EXION

3 MONTHS AFTER THE LAST TREATMENT, COURTESY OF: DIANE DUNCAN, M.D.

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## ● ACI

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## ● LİMİTLİ KULLANIM

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**My face is lobster red, with multiple welts and burns**

Whoever in these reviews says this procedure isn't painful must have an astronomical amount of pain tolerance. I've had in my 3 treatments: 23% lidocaine cream (left on for an hour) as well as lidocaine injections, in addition to nitrous oxide... so all 3 COMBINED for pain... had no effect whatsoever...

**Two weeks post treatment - pain was horrendous and I couldn't sleep**

It's been two weeks since my first treatment on my stomach. The pain was horrendous and I couldn't sleep the first three nights. I've followed post care instructions and after two weeks my abdomen still looks like this! Is this normal two weeks post treatment?

**Not worth the risk - it was extremely horrific and painful**

I wanted to get a little skin tightening around my stomach, my skin was a little loose... I was recommended as a no down time procedure. Firstly even though they numbed the area for 45 minutes, it was extremely horrific and painful. I had to tell the technician to stop towards the...

Source: Realself reviews on competitive devices

# İĞNE DERİNLİK ÖNERİLERİ

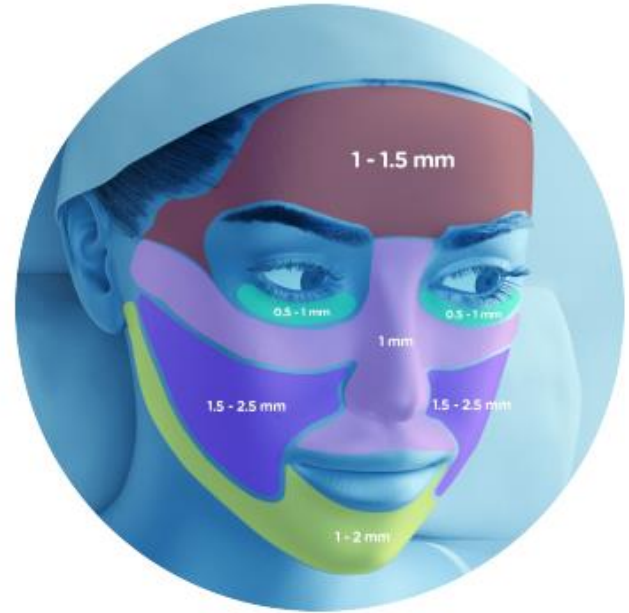
## NEEDLE DEPTH RECOMMENDATION

### Face

- Bony Prominences 0.5 - 1.5 mm
- Periorbital and perioral area 0.5 - 1 mm
- Nose 1 mm
- Neck 0.5 - 1 mm

### Body

- 2 - 4 mm

These parameters are just a recommendation.  
Always adjust the treatment settings based on  
the patient's feedback and skin response.

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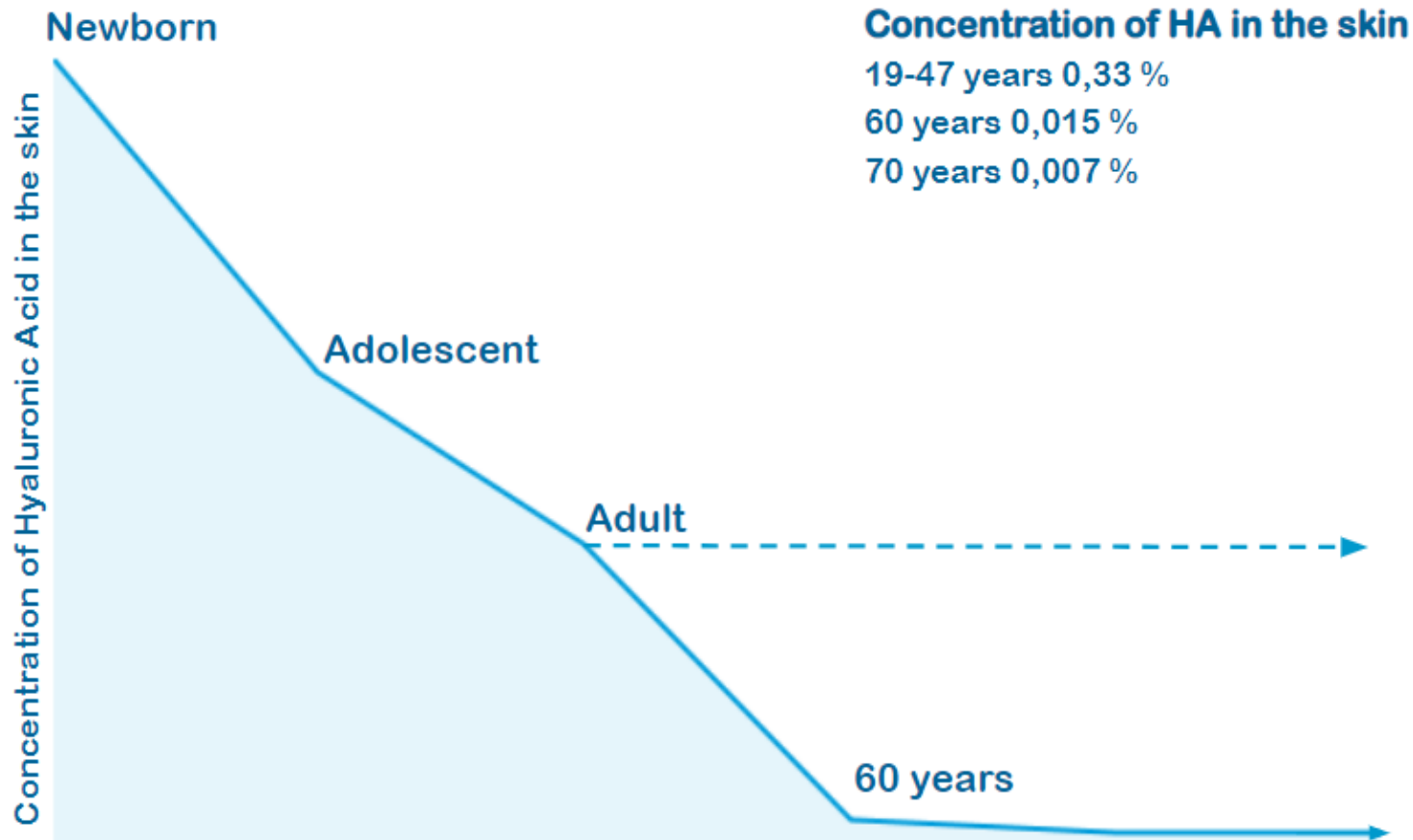
Source: [https://www.researchgate.net/figure/The-decrease-of-epidermal-hyaluronic-acid-content-is-a-hallmark-of-skin-aging-on-the\\_fig7\\_321512333](https://www.researchgate.net/figure/The-decrease-of-epidermal-hyaluronic-acid-content-is-a-hallmark-of-skin-aging-on-the_fig7_321512333)

**The Ultimate Skin Solution**

EXION™

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Today's solutions only deliver HA  
internally or externally:

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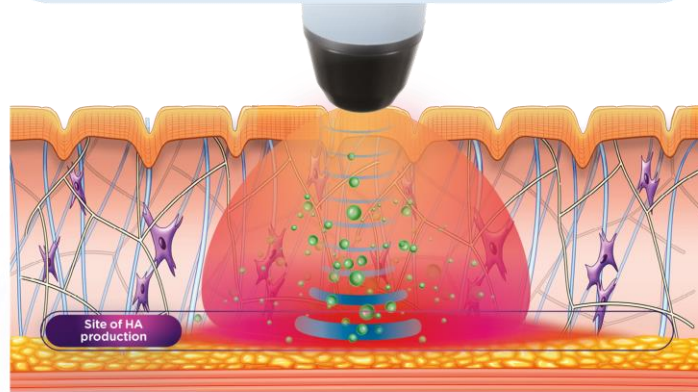
## Simultaneous RF & Targeted Ultrasound

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Increased levels of hyaluronic acid in skin after monopolar radiofrequency and TUS treatment: porcine animal study

Diane Duncan, M.D., FACS, MvDr. Jan Bernardy PhD, MvDr. Nikola Hodkovicova PhD and PharmDr. Josef Masek PhD.

Presented at the American Society for Laser Medicine and Surgery (ASLMS). San Diego, California, 27 April 2022.

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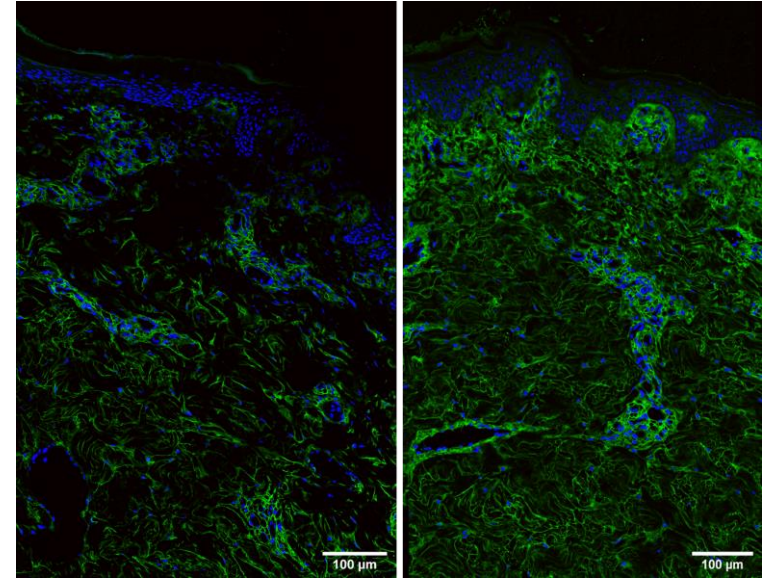
224 %  
Daha  
Fazla HA

RF+ Hedefli Olmayan Ultrason

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*Confocal microscopy images show that at the 2-month follow-up, the network in the dermis of the RF+TUS group appears denser with more green fibers compared to its baseline on the left*

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Radiofrequency And Targeted Ultrasound Have Shown To Enhance Natural Hyaluronic Acid Production: Porcine Animal Study

Klaus Fritz, M.D., MvDr. Jan Bernardy, PhD, MSc. Rea Jarosova, MA. Natalie Kralova.

Presented at the American Society for Laser Medicine and Surgery (ASLMS). San Diego, California, 27 April 2022.

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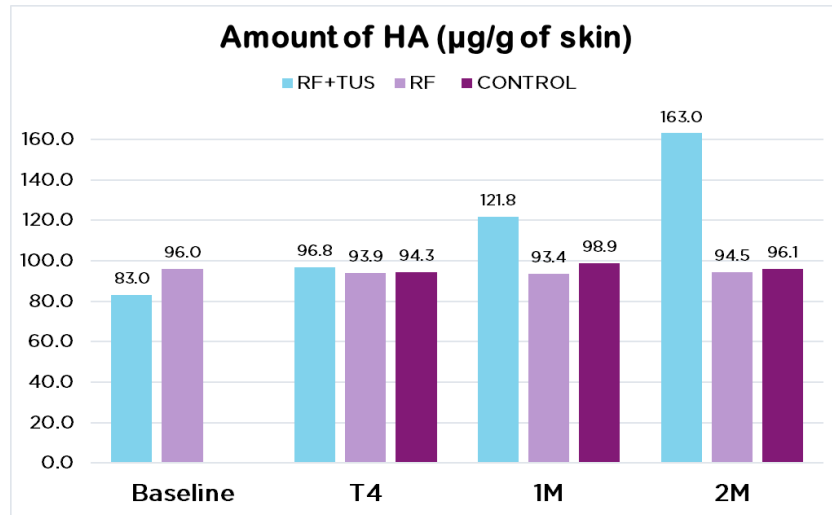
A novel technology to boost natural production of hyaluronic acid in the skin tissue: human histology study.

David J. Goldberg, M.D., J.D.

Presented at the American Society for Laser Medicine and Surgery (ASLMS). San Diego, California, 27 April 2022.

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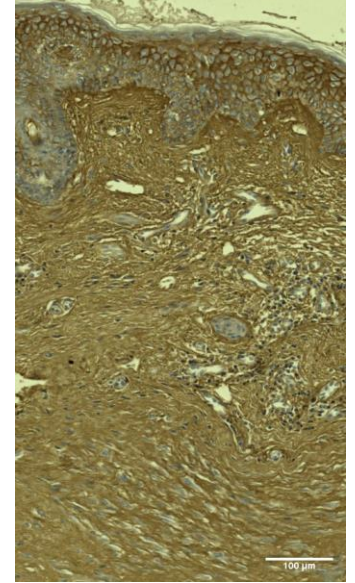
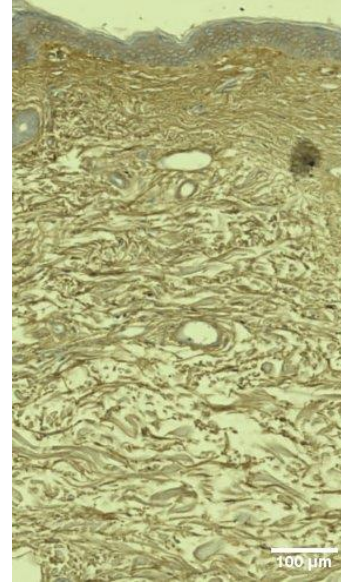
RF+TUS grubu

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*Histology visualizes increased amount of hyaluronic acid (dark brown color) in the RF+TUS group, baseline (left), 3-month follow-up (right).*

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The simultaneous application of monopolar radiofrequency and targeted ultrasound for improvement of skin hydration and skin texture.

Suneel Chilukuri, M.D., FAAD, FACMS; Charles M. Boyd, M.D., MBA, FACS.

Presented at the Annual Meeting of the Vegas Cosmetic Surgery & Aesthetic Dermatology, June 10-12, 2021

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41 %

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23 %

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*52-year-old patient in the RF+TUS group at baseline (left) and at 3-month follow-up (right)*

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
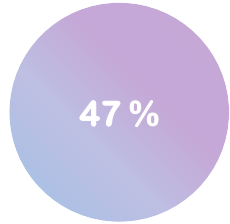
Monopolar radiofrequency and targeted ultrasound induces remodelling of fibrillar collagen and elastin fibers: histological porcine model study  
David E. Kent, M.D.

Presented at the American Society for Laser Medicine and Surgery (ASLMS). San Diego, California, 27 April 2022.

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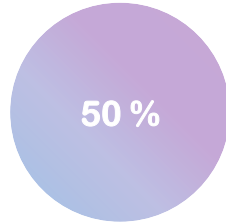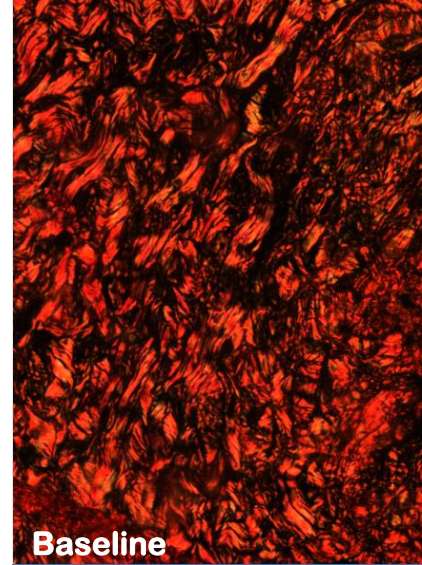
*Overall collagen content increase was significantly higher at 3-month follow-up compared to baseline*

BEFORE

AFTER EXION

3 MONTHS AFTER THE LAST TREATMENT, COURTESY OF: LESLEY CLARK-LOESER, M.D.

**The Ultimate Skin Solution**

EXION™

BEFORE

AFTER EXION

3 MONTHS AFTER THE LAST TREATMENT, COURTESY OF: BTL AESTHETICS

**The Ultimate Skin Solution**

EXION™

BEFORE

AFTER EXION

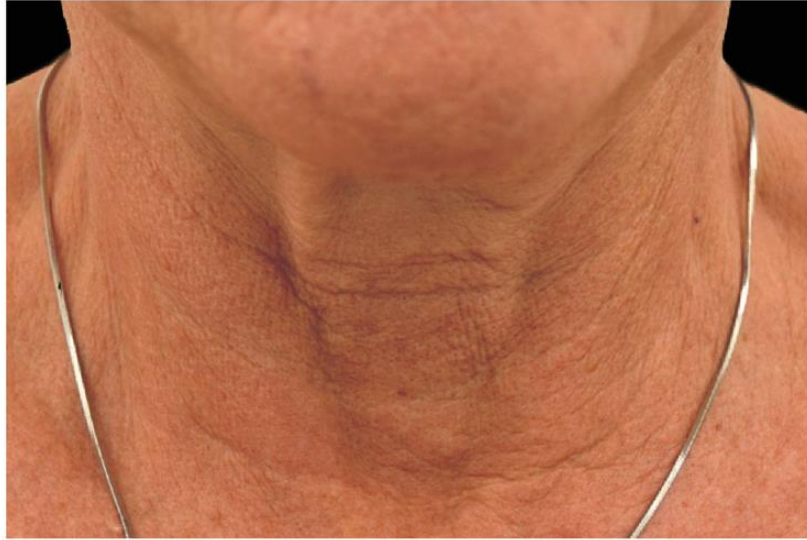
3 MONTHS AFTER THE LAST TREATMENT, COURTESY OF: LESLEY CLARK-LOESER, M.D.

**The Ultimate Skin Solution**

EXION™

BEFORE

AFTER EXION

AFTER THE LAST TREATMENT, COURTESY OF: SUNEEL CHILUKURI, M.D.

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A Novel Technology Combining Radiofrequency and Targeted Ultrasound for improvement in skin laxity: The efficacy and Safety Evaluation

David E. Kent, M.D. , Gregory Mueller, M.D.

Presented at Presented at the American Society for Laser Medicine and Surgery (ASLMS), Phoenix, Arizona, April 13-16 2023

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85 %

Cilt Laksite İyileşmesi

*Improvement in skin laxity on the upper arm of a 60-year-old patient at 3 months follow-up compared to the baseline*

# STUBBORN FAT REDUCTION IN HARD TO REACH AREAS

The efficacy and safety evaluation of a technology using radiofrequency heating with superficial active cooling for localized fat reduction  
Nicole Hayre, M.D., FAAD.

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87%

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*Reduction in fat of a 60-year-old patient  
at the 6 months follow up (right) compared to baseline (left)*

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Increased levels of hyaluronic acid in skin after monopolar radiofrequency and TUS treatment: porcine animal study  
Diane Duncan, M.D., FACS, MvDr. Jan Bernardy PhD, MvDr. Nikola Hodkovicova PhD and PharmDr. Josef Masek PhD.  
Presented at the American Society for Laser Medicine and Surgery (ASLMS). San Diego, California, 27 April 2022.

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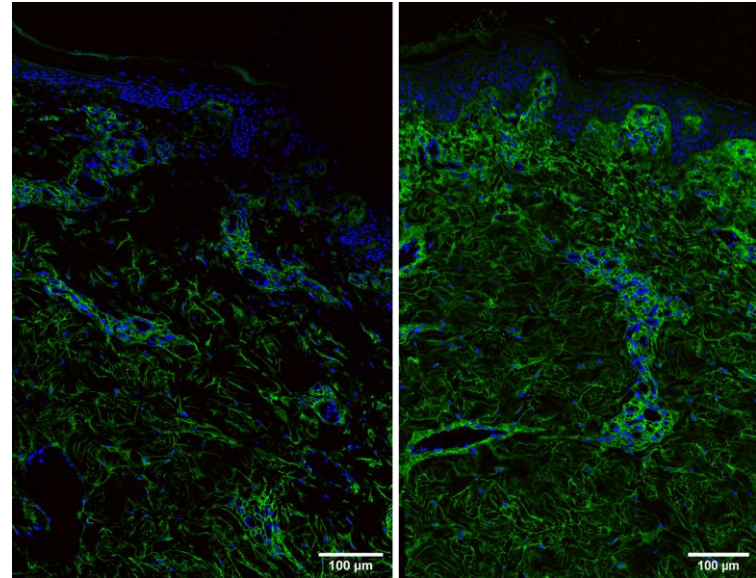
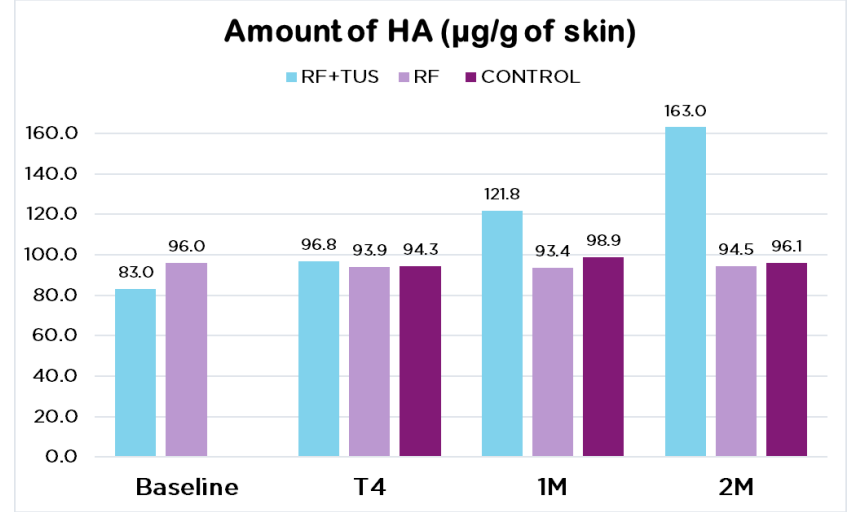
RF+TUS grup

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*The concentration of Hyaluronic Acid was assessed using ELISA kit in samples from three different groups at baseline, after the final treatment, and during the 1-month and 2-month follow-ups.*

BEFORE

AFTER EXION


3 MONTHS AFTER THE LAST TREATMENT, COURTESY OF: DAVID KENT, M.D.

**The Ultimate Skin Solution**

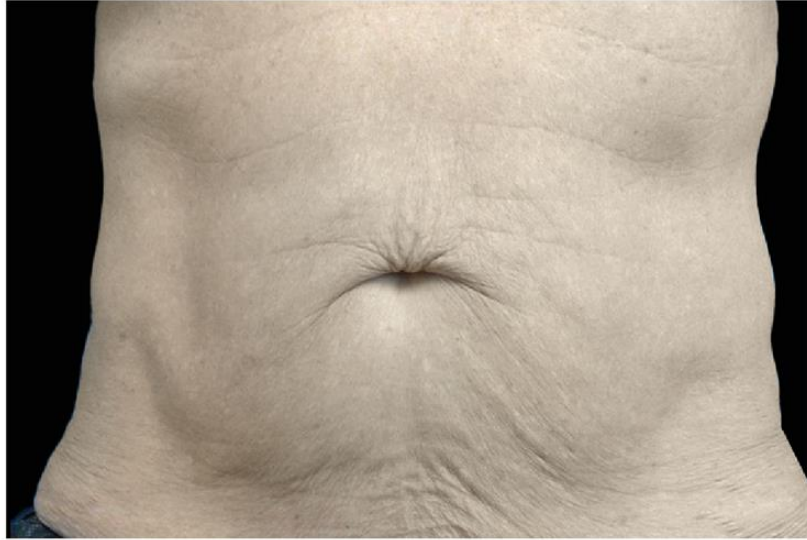EXION™

BEFORE

AFTER EXION

3 MONTHS AFTER THE LAST TREATMENT, COURTESY OF: DAVID KENT, M.D.

**The Ultimate Skin Solution**

EXION™

BEFORE

AFTER EXION

3 MONTHS AFTER THE LAST TREATMENT, COURTESY OF: NICOLE HAYRE, M.D.

**The Ultimate Skin Solution**

EXION™

BEFORE

AFTER EXION

6 MONTHS AFTER THE LAST TREATMENT, COURTESY OF: NICOLE HAYRE, M.D.

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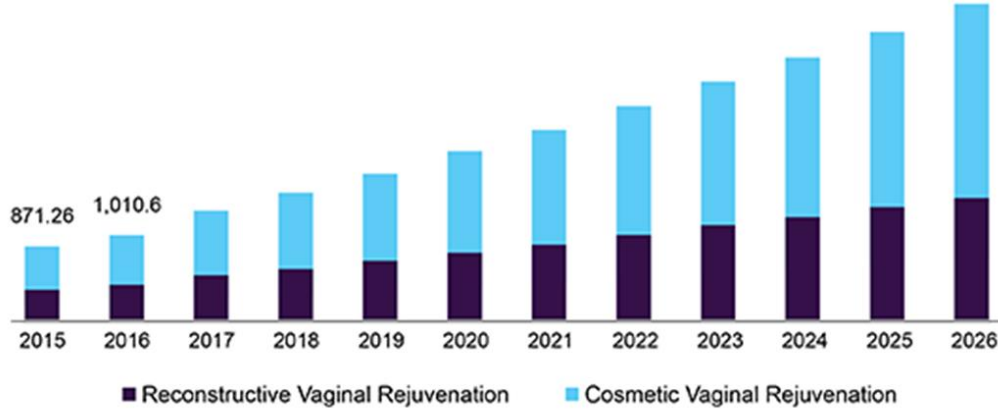
U.S. vaginal rejuvenation market size, by treatment type, 2015 - 2026 (USD Million)

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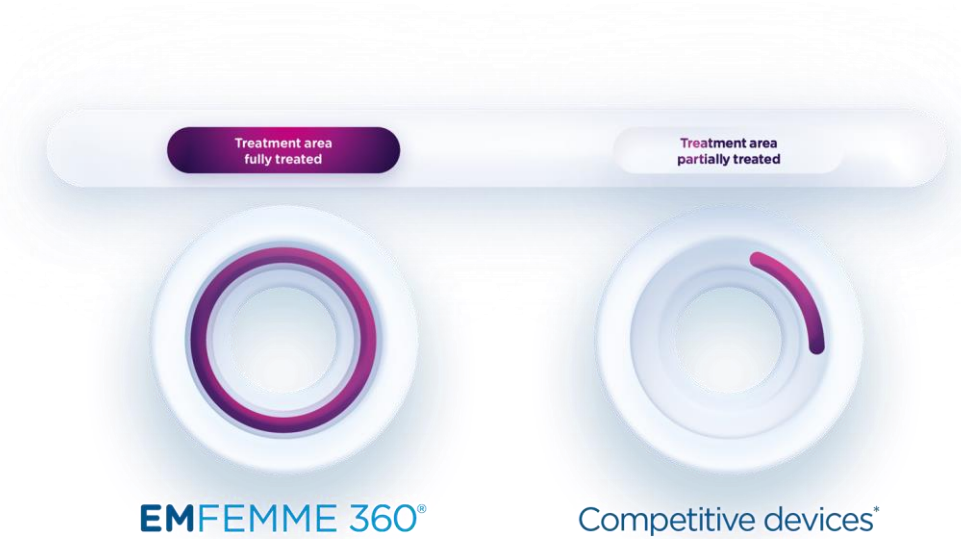
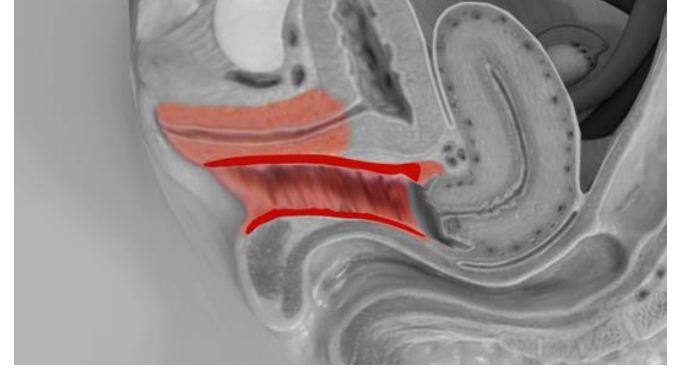
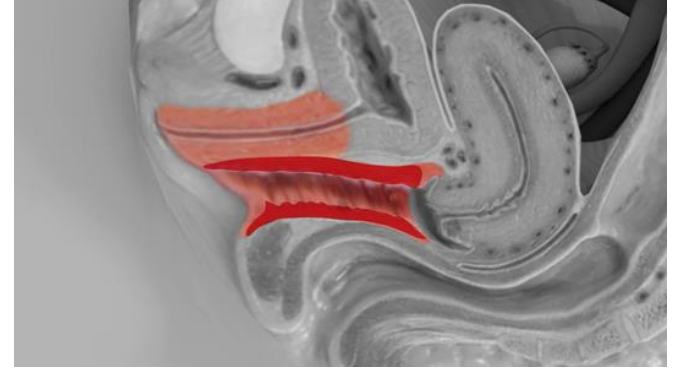
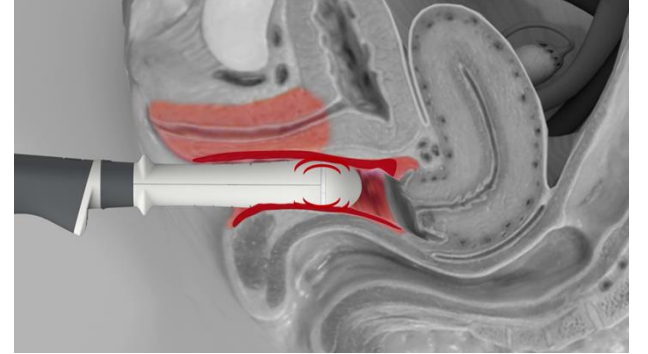
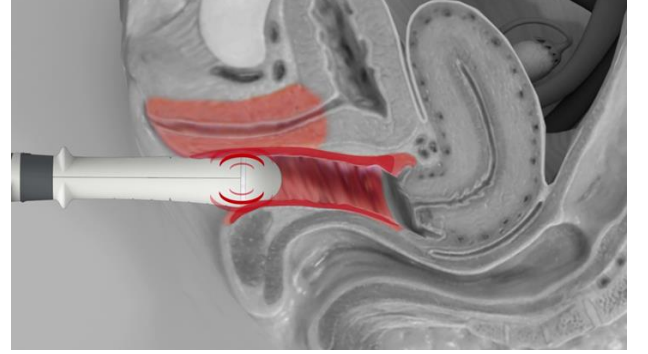
## BACKED BY SCIENCE

### 3 clinical studies:

- **HUMAN HISTOLOGY STUDY - Aznar et al.**
- **VAGINAL HEALTH STUDY - Cheng et al.**

### In progress:

- **VAGINAL MATURATION STUDY - DeLucia et al.**

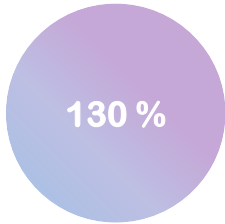
# SHORTEST RADIOFREQUENCY TREATMENT FOR WOMEN'S INTIMATE HEALTH

Assessing The Efficacy of Radiofrequency Treatment for Improving Genitourinary Syndrome of Menopause Symptoms, Sexual Function, And Vaginal Health


Dr H. W. V. Cheng<sup>1</sup>

1. Vitalage, Central, Hong Kong


- 24 postmenopausal subjects (av. 56 years)
- 4 treatments with 1 and 3-month follow-up.
- 3 Different Evaluation Methods
  - Female Sexual Function Index (FSFI)
  - Vaginal Health Index Score (VHIS)
  - GSM Symptoms - Visual Analogue Scale (VAS)

Vaginal health improvement

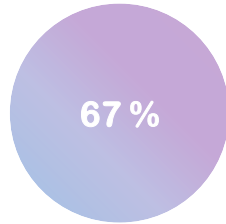Sexual function improvement

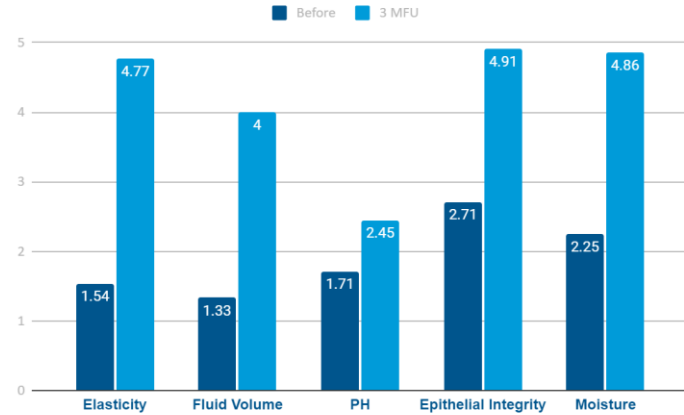*The Vaginal Health Index scores show an overall improvement in all five criteria.*


# RADIOFREQUENCY TECHNOLOGY FOR TREATMENT OF DYSPAREUNIA

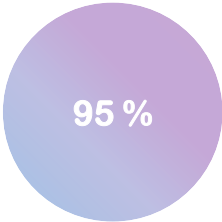360° Radiofrequency Technology for Treatment Of Dyspareunia In Women with Genitourinary Syndrome Of Menopause

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
Published in the Journal of Women's Health Care DOI: 10.35248/2167-0420.22.11.585.


- 35 postmenopausal subjects (av. 56.6 years)
- Three 8-minute treatments, one month apart
- 83% patients (initially incapable) were capable of sexual intercourse after treatments
- Vaginal pH was restored (from 6.6 to a healthy level of 4.6)

Patients reported no discomfort

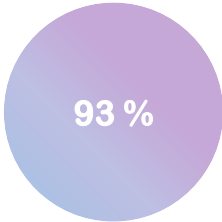Patients were satisfied with results

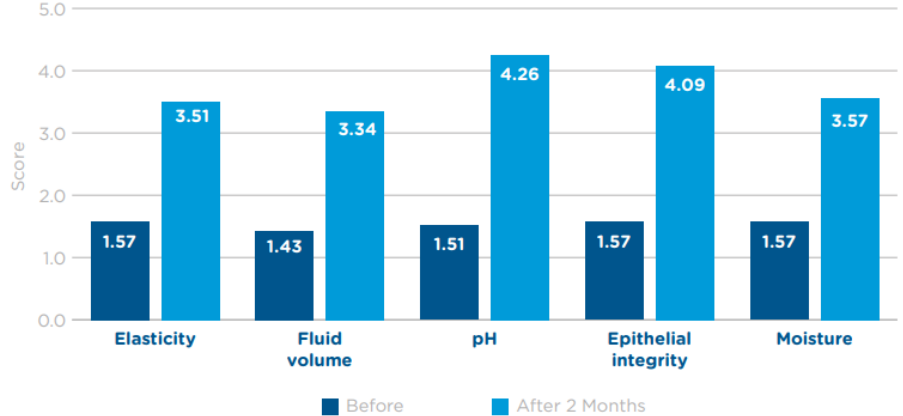The Vaginal Health Index scores show an overall improvement in all five criteria.

# TEŞEKKÜRLER!

The Ultimate Skin Solution

EXION™